



ADAPTER 4A74

SQUARE ADAPTER

SQUARE ADAPTER

aluminum

Square 4-hole adapter with rotation. They are used exclusively for the manufacture of lower extremity prostheses.

Made of high resistance aluminum and stainless steel. Indicated for patients with a maximum weight of 80 kg.

Pyramidal receiving adapter with twist fit, used in combination with any pyramid receiving adapter, for modular leg prostheses.

These combinations provide angle and translation modifications in the sagittal and anterior plane, as well as regulation of internal and external rotation.



ACTIVITY LEVEL



PATIENT WEIGHT LIMIT



IMPACT LEVEL 1-4

RESISTANCE

80 KG

SKU

4A74

UNIT WEIGHT

96 GR

CAUTION! ⚠

Do not expose the prosthetic components to environments that favor the corrosion of metal, as well as the degradation of the elements that make up the adapter. If it is exposed to any of these environments and the adapter has traces of elements that favors its deterioration, the warranty offered by O and P will be lost.

WARRANTY

The 4A74 adapter warranty only applies if it is used under the conditions described in this sheet. This product is insured for 1 year using exclusively O and P modular products.

SQUARE ADAPTER


INSTRUCTIONS FOR USE

Rotary square adapter with rotational adjustment and pyramid receiver.

The union with the opposite piece in the modular system is through the M8 x 14 mm oppressors.¹





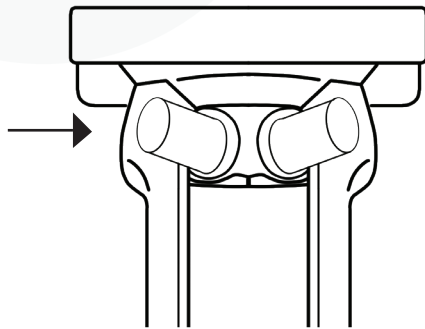



Corrections can be made at any time during alignment, try-in and after completion of the prosthesis. To exchange any part, disassemble it. To keep the previous alignment, unscrew only the two adjoining oppressors.

For the definitive use, secure it with a torque wrench set to 15 NM. The application of Loctite 242 threadlocker is necessary.



1



ACTIVITY LEVEL	   
PATIENT WEIGHT LIMIT	   
IMPACT LEVEL 1-4	RESISTANCE 80 KG
SKU 4A74	UNIT WEIGHT 96 GR