



PROSTHETIC FOOT 1A38

SINGLE AXIS FOOT

SINGLE AXIS FOOT

with shaped toes

The 1A38 O and P single axis foot has a smooth surface and anatomically correct shaped toes, its appearance is natural and discreet.

It is recommended for patients with a degree of mobility of 1 and 2, who carry out activities of moderate impact in mainly closed environments and with mostly flat and regular surfaces. Indicated for patients with a maximum weight of 100 kg.

It is used exclusively for the manufacturing of lower extremity prostheses.



ACTIVITY LEVEL



PATIENT WEIGHT LIMIT



IMPACT LEVEL 1-2

RESISTANCE

100 KG

SKU 1A38

UNIT WEIGHT

Variable*

*The resistance and weight of the product varies according to its size.
Suggested L-Code: L5974

CAUTION!

Do not expose the prosthetic components to environments that favor the corrosion of metal, as well as the degradation of the elements that make up the prosthetic foot. If it is exposed to any of these environments and the foot has traces of elements that favors its deterioration, the warranty offered by O and P will be lost.

WARRANTY

The 1A38 foot warranty only applies if it is used under the conditions described in this sheet. This product is insured for 1 year using exclusively O and P modular products.

INSTRUCTIONS FOR USE



- 1 Insert the tube into the foot adapter.¹
- 2 Tighten the m7 nuts^e with a torque wrench to 6 NM and place the rotation lock^c between the joint of the articulated foot adapter and the foot.²

Corrections can be made at any time during alignment, try-in and after completion of the prosthesis. To exchange any part, disassemble it. To maintain the already adjusted alignment, unscrew only the two adjoining opposors.



IMPORTANT !

It will be necessary to secure the adjustment opposors with Loctite 242 and fix them with a 15 NM torque wrench.



PRODUCT SPECIFICATIONS AND TECHNICAL DATA

BODY WEIGHT	UNIT WEIGHT	SIZE	LEFT/RIGHT	SKU
80 KG	375 GR	22	Izquierdo	1A38=L22
80 KG	375 GR	22	Derecho	1A38=R22
80 KG	380 GR	23	Izquierdo	1A38=L23
80 KG	380 GR	23	Derecho	1A38=R23
100 KG	384 GR	24	Izquierdo	1A38=L24
100 KG	384 GR	24	Derecho	1A38=R24
100 KG	414 GR	25	Izquierdo	1A38=L25
100 KG	414 GR	25	Derecho	1A38=R25
100 KG	420 GR	26	Izquierdo	1A38=L26
100 KG	420 GR	26	Derecho	1A38=R26
100 KG	420 GR	27	Izquierdo	1A38=L27
100 KG	420 GR	27	Derecho	1A38=R27
100 KG	478 GR	28	Izquierdo	1A38=L28
100 KG	478 GR	28	Derecho	1A28=R28
100 KG	583 GR	29	Izquierdo	1A38=L29
100 KG	583 GR	29	Derecho	1A38=R29

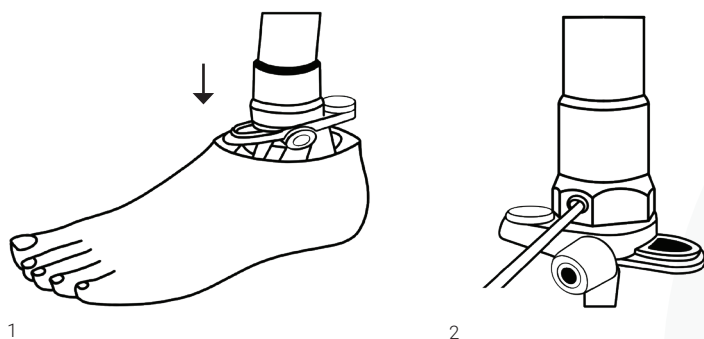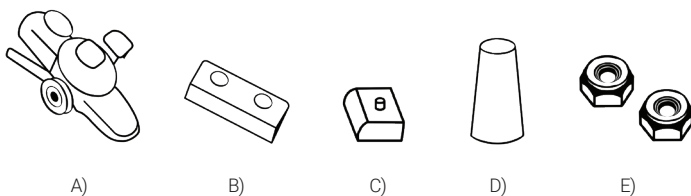

ACTIVITY LEVEL				
PATIENT WEIGHT LIMIT				
IMPACT LEVEL	1-2	RESISTANCE	100 KG	
SKU	1A38	UNIT WEIGHT	Variable*	

* The resistance and weight of the product varies according to its size. Suggested L-Code: L5974