



BK TUBE PROSTHESIS 213

TUBE FOR TRANSFEMORAL PROSTHESIS


TUBE FOR TRANSFEMORAL PROSTHESIS

aluminum

Aluminum tube with stainless steel adapter. They are used exclusively for the manufacturing of lower extremity prostheses.

Made of high resistance stainless steel. Indicated for patients with a maximum weight of 125kg.

The tube has a diameter of 30 mm and a length of 40 cm, not counting the height of the pyramidal receiver at its base.

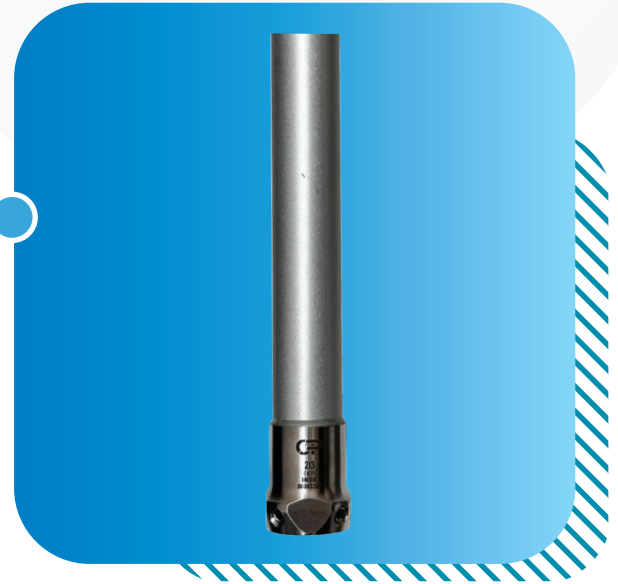

ACTIVITY LEVEL



PATIENT WEIGHT LIMIT



IMPACT LEVEL

1-4

RESISTANCE

125 KG

SKU

213

UNIT WEIGHT

313 GR

CAUTION! ⚠️

Do not expose the prosthetic components to environments that favor the corrosion of metal, as well as the degradation of the elements that make up the tube. If it is exposed to any of these environments and the tube has traces of elements that favors its deterioration, the warranty offered by O and P will be lost.

WARRANTY

The 213 tube warranty only applies if it is used under the conditions described in this sheet. This product is insured for 1 year using exclusively O and P modular products.

TUBE FOR TRANSFEMORAL PROSTHESIS

INSTRUCTIONS FOR USE



The Tube with Adapter 2I3 has a height of 40 cm and is to be cut to the size required by the technician.

Please follow the instructions below to avoid tube deformation. O and P tube tolerances are of approximately .001.

A deformed tube cannot be inserted into a tube adapter (4I21,4I21C, 4I21B, 4A69, 4I56) without further difficulties.


CAUTION!

Do not clamp the tube in the workbench screw, as this action can cause deformations or damage. The cut must form a 90° angle with respect to the outer wall line of the tube in the distal direction. The use of tube cutters (not saws) is recommended.


To install the tube, loosen the M4 x 22 mm screw ^{1.1} from the adapter (4I21), and insert the tube into it until it stops. ^{1.2}

WARNING

For the definitive use of the M4 x 22 mm screw, secure it with a torque wrench set to 13 NM. The application of Loctite 242 threadlocker is necessary. Failure to follow this warning could result in the screw stripping and becoming unusable, as well as loosening over time.

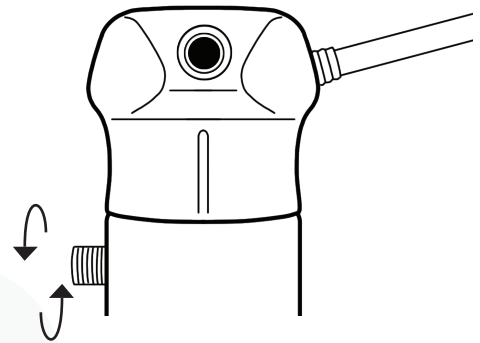

1.1

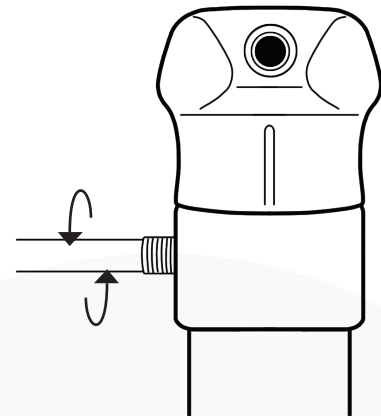

1.2


TUBE FOR TRANSFEMORAL PROSTHESIS

INSTRUCTIONS FOR USE

The union with the opposite piece ² in the modular system is through the M8 x 14 mm oppressors.

Corrections can be made at any time during alignment, try-in and after completion of the prosthesis. To exchange any part, disassemble it. To keep the previous alignment, unscrew only the two adjoining oppressors.

For the definitive use, secure it with a torque wrench set to 15 NM. The application of Loctite 242 threadlocker is necessary.

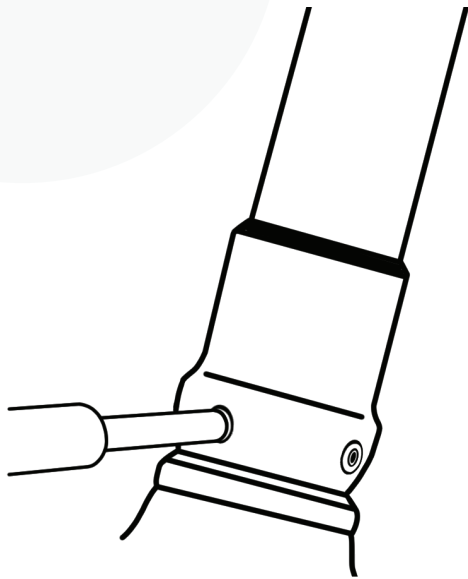

2



ACTIVITY LEVEL



PATIENT WEIGHT LIMIT



IMPACT LEVEL 1-4

RESISTANCE

125 KG

SKU

213

UNIT WEIGHT

313 GR